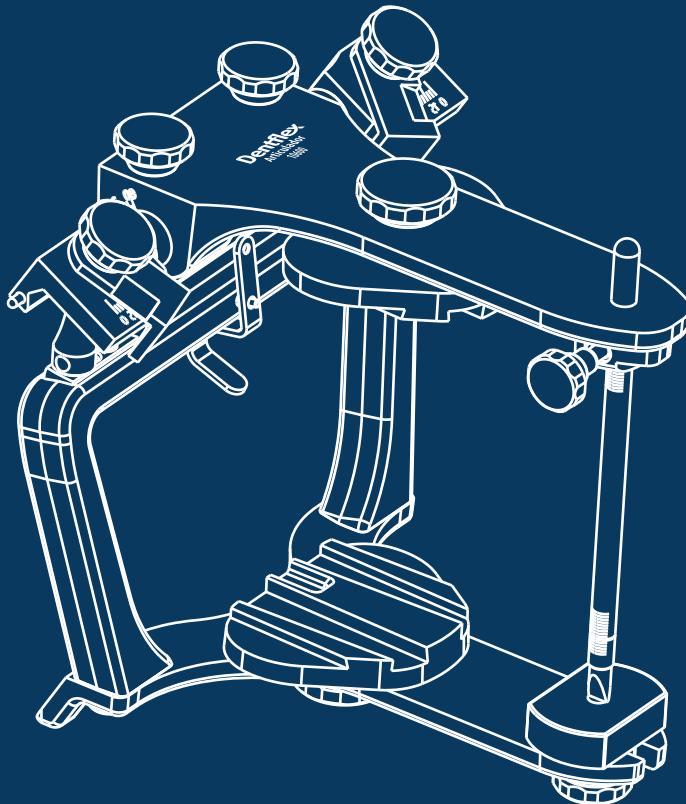
# Dentflex

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## **1. INTRODUCTION**

Congratulations on your purchase of the Articulator 10600.

If you have any questions or problems while reading this handbook instructions, please contact Dentflex. Keep this manual in a safe place.

Dentflex reserves the right to incorporate, at any time and without prior notice, changes in the information and data indicated in this document. This handbook instructions has been developed with absolute rigor. However, even with our efforts, it is impossible to eliminate the possibility of inaccuracies. We are always grateful for any recommendations for improvement, so please contact Dentflex directly.

## **2. DESCRIPTION AND PURPOSES**

The articulator is a mechanical instrument that represents the maxilla, mandible and joint, which allows the assembly of plaster models in the same position in which the jaws are in relation to the condyles and the craniofacial structure, recording and doubling jaw movements.

The semi-adjustable articulator is essential in the diagnosis of occlusal interferences, in the design and manufacture of devices that will be used by the patient as full dentures, fixed and removable partial, interocclusal splints and surgical guides among others. Assists in the treatment plan logical and orderly development.

The Dentflex Articulator 10600 is the Arcom type with strong structural mechanics which guarantee more stability in the opening movements, protrusion and freely and accurately lateralities. Its design provides excellent visibility and ergonomics. The rail mounting plates enable the model removal without breaking the plaster and its magnetic attachment helps in clinical cases' rapid exchange.

The Semi Adjustable Articulator 10600 does not offer interchangeability, that is, the models mounted on an articulator must not be transferred (assembled) to any other articulator.

### 3. ACCESSORIES

#### Standard Face Bow

The face bow is an instrument used to record the spatial position of the upper dental arch of the patient in relation to the skull and transfer this record to the articulator

#### Case

Case used for the Dentflex articulator's transportation.

#### Toothless Fork

Product used for total edentulous patients' registration with Face Bow. It was developed and designed through clinical tests carried out at the São Paulo University - Bauru Faculty of Dentistry by Prof. Dr. Conti.

#### Metallic Adjustable Incisal Table

Product used in the teeth balancing process, allowing the anterior guide customization through its lateral walls and antero-posterior plane, both adjustable.

#### Camper Plan With 15° Inclination

Support used for the upper model's standard assembly in the articulator.

#### Fox Ruler

Product indicated for the orientation planes' determination during total dentures' assembly.

#### Fork Support

This product is used as a support during the upper model assembly when it is performed with a face bow.

Obs.: The above accessories are not included with the product and can be purchased separately.

## 4. HANDBOOK INSTRUCTIONS

### 4.1 Registration With the Face-Bow Standard

Fig. 01

Fig. 02

Fig. 03

Fig. 04

Fig. 05

With the registration material Godiva type, choose 3 points in the toothless fork, 2 in the posterior region in each hemiarch and 1 in the anterior region (fig. 01). Upon at the upper arch registration, center the fork stem with average facial line of the patient and press on the occlusal surface of the upper teeth until the material lamination (fig. 02). Try the plaster model on registration material to confirm its stability (fig. 03).

Unscrew the face bow universal stems to have a good slip of its components and, attach and tighten the nazium fixing screw at the crossbar (fig. 04). Return the fork with the bite registration in the correct position in the mouth and ask to the patient to maintain pressing the bottom of the fork with the thumbs bilaterally against the jaw (fig. 05). Place the face bow introducing the fixing assembly with universal joints on the bite fork rod and attach the plastic atria at the external auditory canal and tighten the 3 fixing screws of the face bow lateral branches, not to open the set. Support and press the nazium on the patient forehead, tightening the fixing screw of the nazium rod (fig. 06). With nazium and face-bow placed, first tighten the universal joint of the vertical rod with a slight finger pressure in this set up (fig. 07), and then we must secure the universal joint of the horizontal rod taking care to leave more near to the fork (fig. 08). NOTE: Tighten the universal joints to prevent displacement.

Before removing the face bow you should check that the bite fork registration mold is perfectly adapted to the occlusal and incisal of the upper teeth. If it is not adapted correctly you need to repeat the installation.

To remove the face bow you need to loosen the nazium set screw and the side rods' central screw; remove the whole set carefully. The universal joints are sensitive and we must take care not touching the universal joints next to the fork.

Fig. 06

Fig. 07

Fig. 08

## 4.2 Intercondylar Distance Adjustment

1 - Put the condylar elements in the lower branch holes corresponding to the intercondylar distance obtained with the face bow at the time of recording the points (S, M or L). Using the lever wrench (condylar stem), lightly tighten the condylar elements. (fig. 09)

Fig. 09

2 - Establish the same intercondylar distance in the articulator's upper branch, expanding or closing the condylic guides using the spreading bushings. (fig. 10)

Fig. 10

The relationship between intercondylar distances and condylic guides is as follows:

**Small point registration (S):** Remove all the thickening bushings, that is, leave the articulator without expansion; (fig. 11)

**Medium point registration (M):** Remove the first thickening bushings on both sides, that is, remove the bushings that have no grooves; (fig. 12).

**Large point registration (L):** Keep all the thickening bushings; (fig. 13).

Fig. 11

Fig. 12

Fig. 13

### WARNING

- Make sure that the device is adjusted so that there is no lateral movement of the upper branch in relation to the lower branch, and observe the position of the condylar elements that should touch the posterior and upper walls of the Bennett's condylic and lateral guides simultaneously. (fig. 14)

Fig. 14

3 - Adjust the condyllic guides with an angle of 30° and the lateral angle (BENNETT) negative (figs. 15 and 16), so that the articulator is stable in the central position. Then, secure the spring handle to the lower branch pin. (fig. 17)

Fig. 15

Fig. 16

Fig. 17

### 4.3 Superior Model Set

1 - Take the face bow attached to the articulator's upper branch through the small pins located on the external face of the condyllic guides, fitting them into the holes of the augivas. (fig. 18)

2 - Leave the front part of the articulator's upper branch supported on the bow's crossbar. Close the face bow securely, tighten its central screw and place the arch and upper branch assembly over the articulator's lower branch (figs. 19 and 20).

3 - Position the upper plaster model, with retentions and previously hydrated, on the fork register. Lift the articulator's upper branch and place a plaster's portion on top of the model. Also place a small amount of plaster on the upper branch mounting plate. With one hand keep the fork and the model in position, to avoid any untimely movement, and close the articulator until the upper branch touches the face bow's crossbar. Wait until the plaster solidifies. (fig. 21)

4 - Remove the face bow of articulator.

Fig. 18

Fig. 19

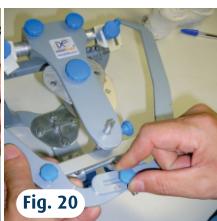
Fig. 20

Fig. 21

#### NOTE

Avoid dropping plaster at the junction of the mounting plate with the articulator branch.

#### WARNING

- To obtain stability and precision during the transfer, use the fork support (optional) and plaster type IV.

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Fig. 22

2 - Then, place the articulator upside down on the laboratory table and seat the lower model, previously hydrated and with retentions, on the interocclusal register, which must be fitted on the assembled upper model. In the total edentulous, this must be achieved by joining the wax rollers.

3 - We recommend that the models be secured with rubber bands or toothpicks (fixed with sticky wax), so that they remain in position while the plaster hardens. Place a portion of plaster on the model and a small amount on the mounting plate of the articulator's lower branch.

4 - Fit the condylar elements to the condylar guides and close the articulator until the incisal pin touches the incisal table. Then, fix the articulator branches with an elastic band to prevent possible distortions caused by the expansion of the plaster.

### NOTE

Avoid dropping plaster at the junction of the mounting plate with the articulator branch.

5 - After the plaster solidifies, place the articulator in its normal position (lower branch supported on the laboratory table) and finish the assembly, filling the fixation towers of the models with plaster to improve the finish. (fig. 23)

Fig. 23

## 4.4 Rail Mounting Plate

The rail mounting plates allow the model removal without breaking the plaster and the possibility of these models reassembly if you need a case's review.


To remove boost the plaster to the left toward next to the rail mounting plate biggest side, as you can see at the image. (fig 24)


Fig. 24

## 5. PIECES' LIST
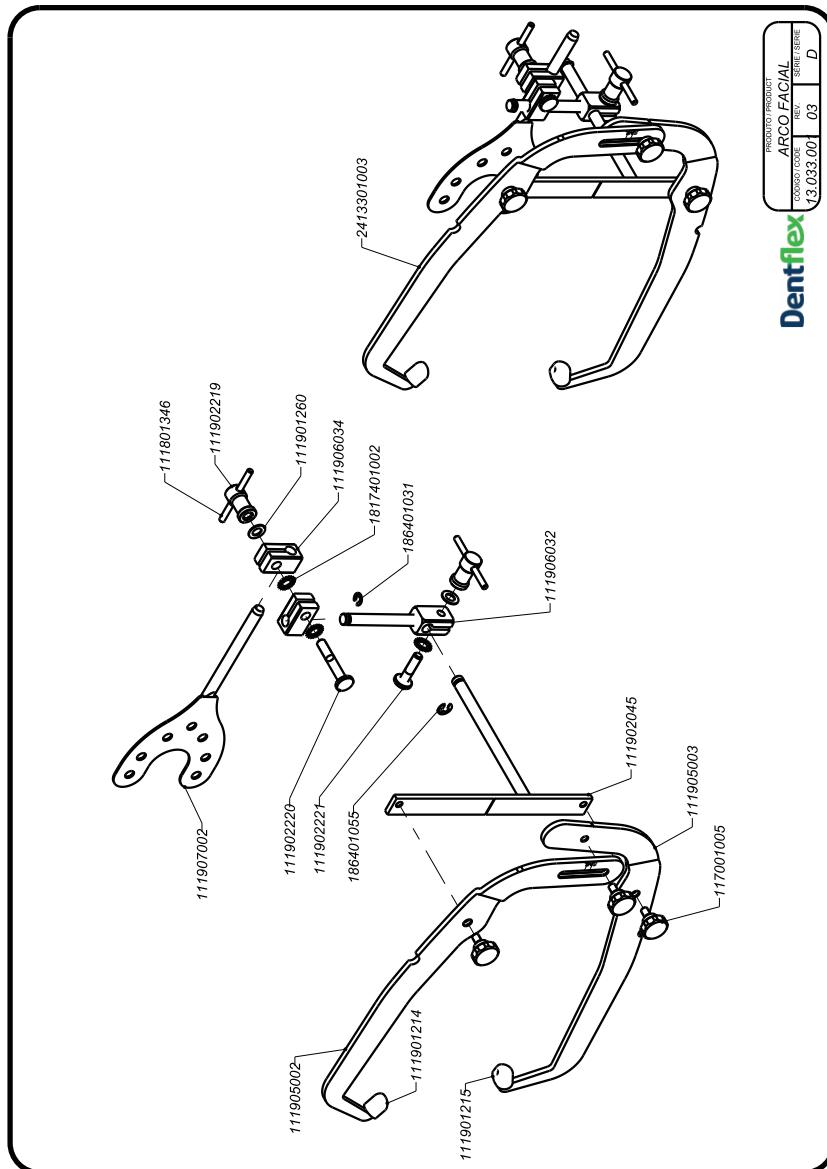
### 5.1 Face-Bow Standard

## 5.2 Articulator 10600

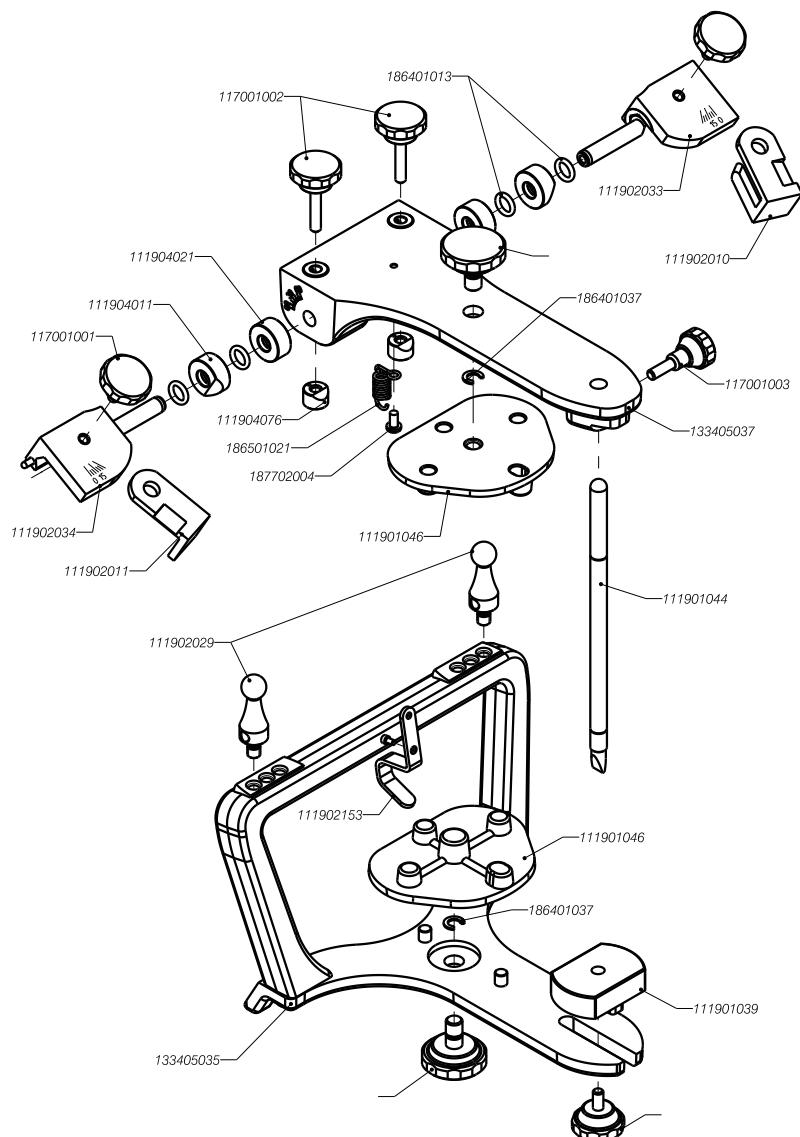**Dentflex**

PRODUTO / PRODUCT ARTICULADOR 10600	CÓDIGO / CODE 15.099.001	REV. 01	SÉRIE / SÉRIE C
--	-----------------------------	------------	--------------------

## 6. PRECAUTIONS AND CONSERVATION

Dentflex recommends only warm water for general cleaning of the articulator.

Before and after using the face bow, it should be cleaned with warm water, and the fork should be sterilized in an autoclave at a maximum temperature of 135 ° C and 2.2 bar, and augie should be disinfected with a handkerchief or bactericidal alcohol 70 ° .

Sterilize the equipment before using it for the first time.

## 7. CORRECTIVE MAINTENANCE

All recommendations for use are found in this Manual, but if any problems or malfunctions are detected please contact the Dentflex Technical Support Center to correct the problem. Do not attempt to open the product and / or repair it yourself or with help of persons not trained / authorized to do so, this may aggravate the problem or even generate faults that could compromise product safety.

## 8. WARRANTY

For guarantee's information see the certificate of the product. The warranty product covers manufacture defects counted from the date of sale to the first client or the delivery of the warranty certificate.

The guarantee becomes invalid in case any of the following situations is verified:

- Installation and/or operation in disagreement with this manual;
- Fall;
- Damages to the external finishing and/ or accessories whose the manufacturer has not given any cause.

## 9. DENTFLEX TECHNICAL SUPPORT CENTER

With the purpose of offering a service that guarantees 100% satisfaction, we present to the customer the Dentflex Technical Support Center, an innovation in the service that came to provide agility, safety and transparency in the maintenance process of your Dentflex product.

We have an infrastructure and a qualified professionals' team, guaranteeing the most complete technical assistance for the entire Dentflex product line.

- Trained technicians and engineers;
- Last generation machinery and equipment;
- Original components.

To use the service offered by the Dentflex Technical Support Center, please, contact us:  
E-mail: [sac@dentflex.com.br](mailto:sac@dentflex.com.br) / Phones: + 55 (16) 9 9404-4435 and + 55 (16) 2133-0000.

**ANVISA / MS Registry:  
FREE  
Technical Manager:  
Engineer Alan Trevizani Chiappa  
CREA / SP n°: 5061315280**

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