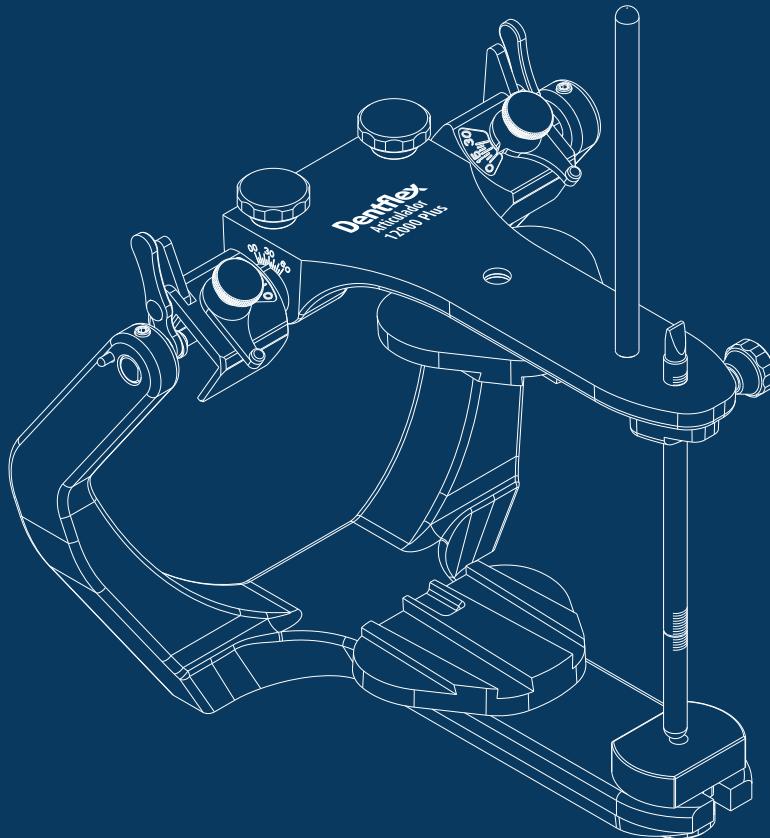
Dentflex

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REVISÃO: 01

Summary

1. INTRODUCTION.....	14
2. DESCRIPTION AND PURPOSES.....	14
3. ACCESSORIES.....	15
4. HANDBOOK INSTRUCTIONS.....	16
4.1 Registration With the Face-Bow Standard.....	16
4.2 Upper Model Assembly.....	17
4.3 Lower Model Assembly.....	18
4.4 Rail Mounting Plate.....	18
5. PIECES' LIST.....	19
5.1 Face-Bow Standard.....	19
5.2 Articulator 12000 Plus.....	20
6. PRECAUTIONS AND CONSERVATION.....	21
7. CORRECTIVE MAINTENANCE.....	21
8. WARRANTY.....	21
9. DENTFLEX TECHNICAL SUPPORT CENTER.....	22
INSTRUCCIONES DE USO - ESPAÑOL.....	23

1. INTRODUCTION

Congratulations on your purchase of the Articulator 12000 Plus.

If you have any questions or problems while reading this handbook instructions, please contact Dentflex. Keep this manual in a safe place.

Dentflex reserves the right to incorporate, at any time and without prior notice, changes in the information and data indicated in this document. This handbook instructions has been developed with absolute rigor. However, even with our efforts, it is impossible to eliminate the possibility of inaccuracies. We are always grateful for any recommendations for improvement, so please contact Dentflex directly.

2. DESCRIPTION AND PURPOSES

The articulator is a mechanical instrument that represents the maxilla, mandible and joint, which allows the assembly of plaster models in the same position in which the jaws are in relation to the condyles and the craniofacial structure, recording and doubling jaw movements. The semi-adjustable articulator is essential in the diagnosis of occlusal interferences, in the design and manufacture of devices that will be used by the patient as full dentures, fixed and removable partial, interocclusal splints and surgical guides among others. Assists in the treatment plan logical and orderly development.

The Dentflex Articulator 12000 Plus is the Arcom type with fixed intercondylar distance and has strong structural mechanics which guarantee more stability in the opening movements, protrusion and freely and accurately lateralities. Its design provides excellent visibility and ergonomics. The rail mounting plates enable the model removal without breaking the plaster and its magnetic attachment helps in clinical cases' rapid exchange. The condylar guides are curved for a more faithful reproduction of the human anatomy. They possess central lock for branches' alignment and enable the opening 180 degrees with the block support in the open position. It allows the angles of the condylar guides and Bennett adjustment and it's immobilization system for the movement is with silicone joint.

3. ACCESSORIES

Standard Face Bow

The face bow is an instrument used to record the spatial position of the upper dental arch of the patient in relation to the skull and transfer this record to the articulator

Case

Case used for the Dentflex articulator's transportation.

Toothless Fork

Product used for total edentulous patients' registration with Face Bow. It was developed and designed through clinical tests carried out at the São Paulo University - Bauru Faculty of Dentistry by Prof. Dr. Conti.

Metallic Adjustable Incisal Table

Product used in the teeth balancing process, allowing the anterior guide customization through its lateral walls and antero-posterior plane, both adjustable.

Camper Plan With 15° Inclination

Support used for the upper model's standard assembly in the articulator.

Fox Ruler

Product indicated for the orientation planes' determination during total dentures' assembly.

Fork Support

This product is used as a support during the upper model assembly when it is performed with a face bow.

Obs.: The above accessories are not included with the product and can be purchased separately.

4. HANDBOOK INSTRUCTIONS

4.1 Registration With the Face-Bow Standard

Fig. 01

Fig. 02

Fig. 03

Fig. 04

Fig. 05

With the registration material Godiva type, choose 3 points in the toothless fork, 2 in the posterior region in each hemiarch and 1 in the anterior region (fig. 01). Upon at the upper arch registration, center the fork stem with average facial line of the patient and press on the occlusal surface of the upper teeth until the material lamination (fig. 02). Try the plaster model on registration material to confirm its stability (fig. 03).

Unscrew the face bow universal stems to have a good slip of its components and, attach and tighten the nazium fixing screw at the crossbar (fig. 04). Return the fork with the bite registration in the correct position in the mouth and ask to the patient to maintain pressing the bottom of the fork with the thumbs bilaterally against the jaw (fig. 05). Place the face bow introducing the fixing assembly with universal joints on the bite fork rod and attach the plastic atria at the external auditory canal and tighten the 3 fixing screws of the face bow lateral branches, not to open the set. Support and press the nazium on the patient forehead, tightening the fixing screw of the nazium rod (fig. 06). With nazium and face-bow placed, first tighten the universal joint of the vertical rod with a slight finger pressure in this set up (fig. 07), and then we must secure the universal joint of the horizontal rod taking care to leave more near to the fork (fig. 08). NOTE: Tighten the universal joints to prevent displacement.

Before removing the face bow you should check that the bite fork registration mold is perfectly adapted to the occlusal and incisal of the upper teeth. If it is not adapted correctly you need to repeat the installation.

To remove the face bow you need to loosen the nazium set screw and the side rods' central screw; remove the whole set carefully. The universal joints are sensitive and we must take care not touching the universal joints next to the fork.

Fig. 06

Fig. 07

Fig. 08

4.2 Upper Model Assembly

AVISO

- Adjust the condylar guides angle at 30° and Bennett at 15° tightening the screws gently.
- Push the latch back so that the branches are aligned.

Adapt the Face Bow with the registration of the jaw position in the articulator and without naziun. Place the holes in the plastic atria that are at the ends of lateral branches on the side dowel pins of the articulator with the upper frame lifted (fig. 09), after lower the upper frame until it touches the transverse rod of the articulator and hit the center screw fixation of lateral stems (fig. 10). Then we put the upper model of the patient with retentions and previously moistened on the bite fork registration, always observing the correct position (fig. 11). Place the optional fork holder on the lower branch of the articulator for better stability of the set (fig. 12).

Fig. 09

Fig. 10

Fig. 11

Fig. 12

With the articulator opening in 180 degrees using the bearing pin which is screwed into the upper articulator branch should manipulate a gypsum rock type following the manufacturer's directions and put an appropriate portion above the upper model and part of the mixture of gypsum in the retention of the mounting plate in sufficient quantity (fig. 13). Close the articulator until the upper branch touch the face bow crossbar and wait for plaster solidification (fig. 14).

After setting plaster remove the face bow releasing the central screw of the side rods and thus opening the rods for withdrawal.(fig. 15).

Fig. 13

Fig. 14

Fig. 15

4.3 Lower Model Assembly

Remove the fork support bracket and adjust the mounting plate in the lower branch, place the incisal block on the upper branch of the articulator with the pointed tip up and the round tip in the middle of the incisal guide table. At the top there is a continuous path that surrounds around the pin, the trace must be in the upper position of the branch, thereby enabling the upper frame is parallel with the lower branch (zero mark).

Must be mounted the lower model after the registration between inter maxillary relationship of the patient who may be in a centric relation position or acquired centric occlusal position, according to the clinical case.

With the inverted articulator on the table with the bearing pin which is screwed into the upper branch is placed in inter maxillary relationship registration on the upper model mounted (fig. 16) and relates it to the lower model fastened with elastic (fig. 17) or other method. Is placed in a portion of plaster on the basis of previously made lower model holds and hydrated gypsum and the retention of the mounting plate of the lower branch (fig. 18). Close the articulator until the pin incisal guide touches the incisal guide table and secure the branches of the articulator with a rubber band to prevent possible distortions caused by the expansion of the plaster (fig. 19).

Fig. 16

Fig. 17

Fig. 18

Fig. 19

After setting of the gypsum plaster, you should add it between the models and the mounting plate and then to finish with sandpaper for final use.

4.4 Rail Mounting Plate

The rail mounting plates allow the model removal without breaking the plaster and the possibility of these models reassembly if you need a case's review.


To remove boost the plaster to the left toward next to the rail mounting plate biggest side, as you can see at the image. (fig 20)



Fig. 20

5. PIECES' LIST

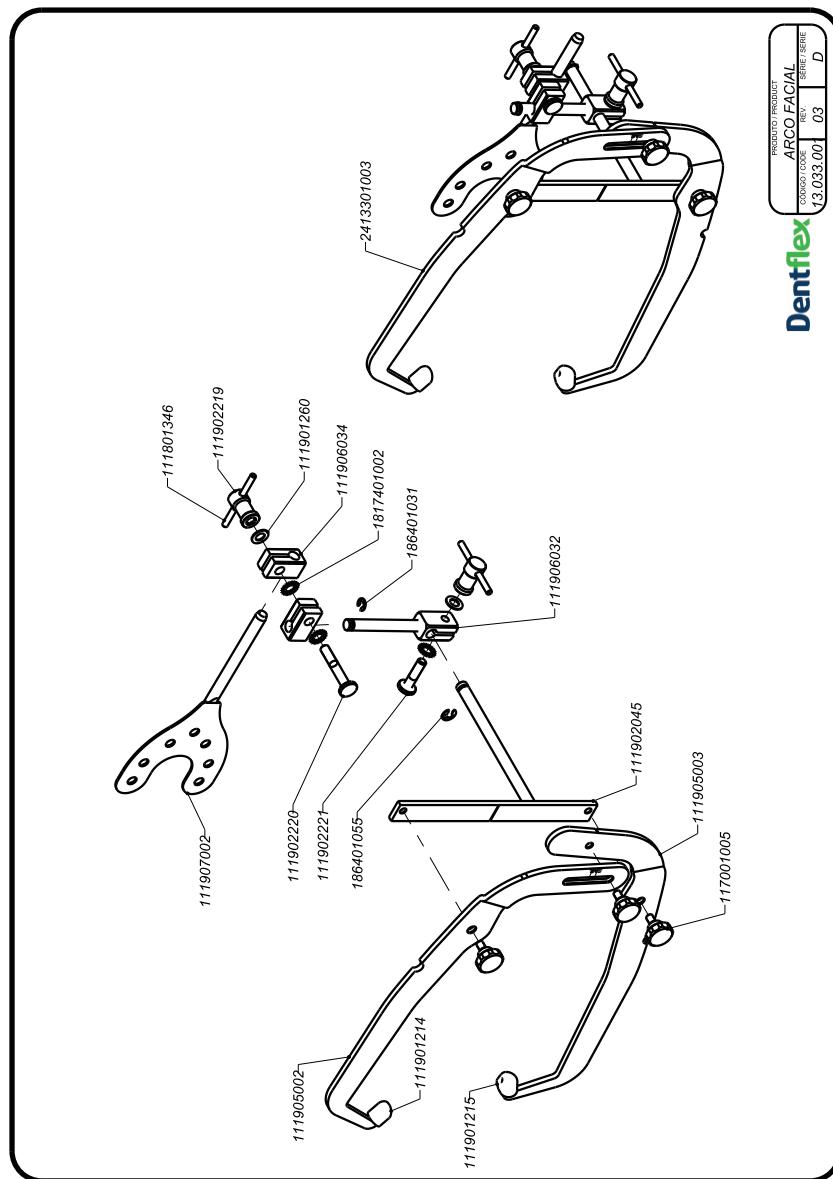5.1 Face-Bow Standard

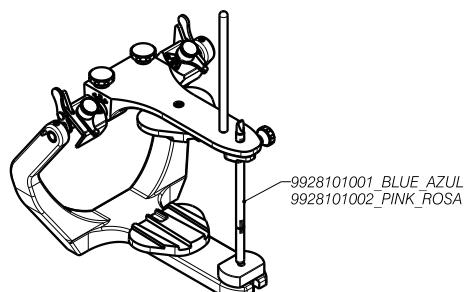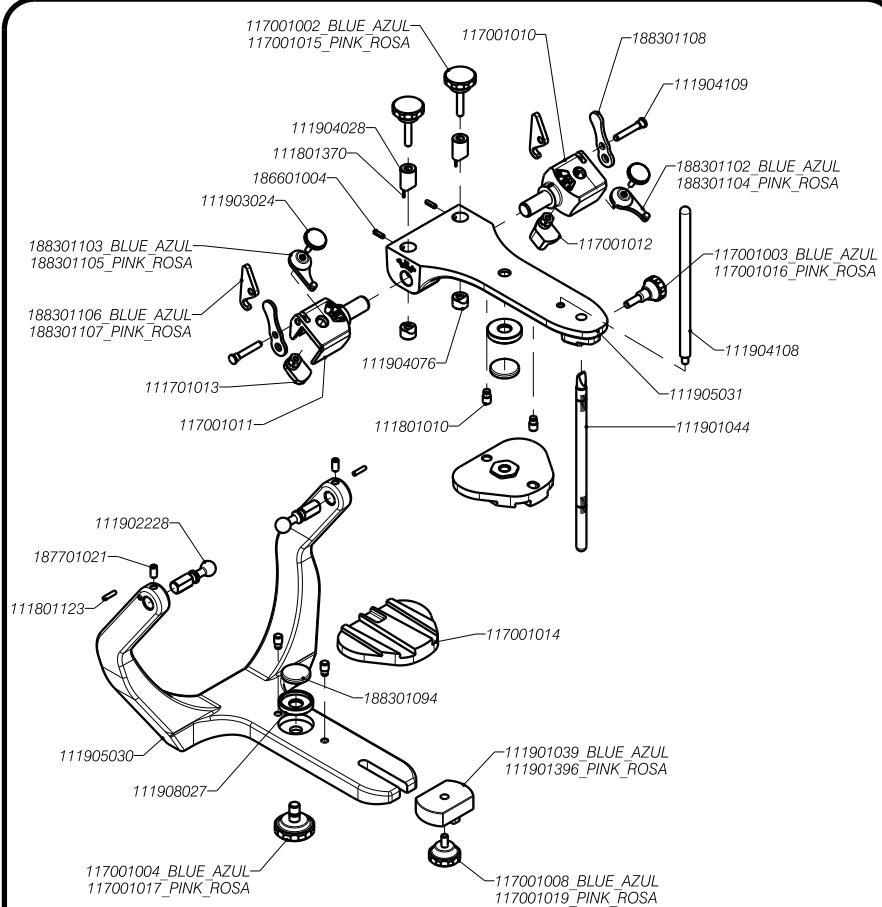5.2 Articulator 12000 Plus

PRODUTO / PRODUCT
Dentflex ARTICULADOR 12000 PLUS
CODIGO / CODE REV. SERIE / SERIE
01 00 A

6. PRECAUTIONS AND CONSERVATION

Dentflex recommends only warm water for general cleaning of the articulator.

Before and after using the face bow, it should be cleaned with warm water, and the fork should be sterilized in an autoclave at a maximum temperature of 135 ° C and 2.2 bar, and augie should be disinfected with a handkerchief or bactericidal alcohol 70 ° .

Sterilize the equipment before using it for the first time.

7. CORRECTIVE MAINTENANCE

All recommendations for use are found in this Manual, but if any problems or malfunctions are detected please contact the Dentflex Technical Support Center to correct the problem. Do not attempt to open the product and / or repair it yourself or with help of persons not trained / authorized to do so, this may aggravate the problem or even generate faults that could compromise product safety.

8. WARRANTY

For guarantee's information see the certificate of the product. The warranty product covers manufacture defects counted from the date of sale to the first client or the delivery of the warranty certificate.

The guarantee becomes invalid in case any of the following situations is verified:

- Installation and/or operation in disagreement with this manual;
- Fall;
- Damages to the external finishing and/ or accessories whose the manufacturer has not given any cause.

9. DENTFLEX TECHNICAL SUPPORT CENTER

With the purpose of offering a service that guarantees 100% satisfaction, we present to the customer the Dentflex Technical Support Center, an innovation in the service that came to provide agility, safety and transparency in the maintenance process of your Dentflex product.

We have an infrastructure and a qualified professionals' team, guaranteeing the most complete technical assistance for the entire Dentflex product line.

- Trained technicians and engineers;
- Last generation machinery and equipment;
- Original components.

To use the service offered by the Dentflex Technical Support Center, please, contact us:
E-mail: sac@dentflex.com.br / Phones: + 55 (16) 9 9404-4435 and + 55 (16) 2133-0000.

**ANVISA / MS Registry:
FREE
Technical Manager:
Engineer Alan Trevizani Chiappa
CREA / SP n°: 5061315280**

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