# Dentflex

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*The low speed is a transmission equipment, when coupled to the air motor has the aim to fix and rotate burs used in many dental procedures.*

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*The equipment is used in dental treatments and for the confection of dental prothesis.*

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*The Low Speed is designated for dental procedures and can only be used and installed for a qualified and capacitated professional on dental area. Its use for any other purpose is not allowed. Handling it improperly constitutes a danger source.*

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*The equipment can be disinfected with alcohol 70° GL, applied on its surface with a cloth or spray.*

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*Do not clean the equipment using ultra sonic cleaners*

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*The equipment must be sterilized before and after each treatment.*

*Pack the product in packaging for sterilization and dispose only in an autoclave at a maximum temperature 135° C and 2.2 Bar.*

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*Sterilize the equipment before using it for the first time. Use compressed dry air, filtered and with no oil.*

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*Keep in a place protected from the sun, heat, dust and humidity;*

*Keep away from chemicals and cleaning agents;*

*Carry out the transport only in its respective packaging;*

*Ambient temperature (transport and storage): -10°C to 55°C;*

*Relative air humidity (transport and storage): 0% to 85%;*

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1 - Fit the air motor to the unit's terminal observing the ducts direction (Fig. 1). Screw the unit's terminal extremity to get fixed to the equipment (Fig. 2).

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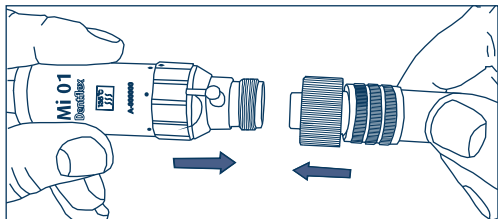
Fig. 1 - Encaixe do Micromotor ao Terminal do Equipó.

Fig. 1 - Fit the Air Motor to the Unit's Terminal.

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*The dental unit's terminal must be manufactured in accordance with ISO-9168 norm, type A connector*

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*1 - Press the Contra Angle over the Air Motor coupling until it gets fixed. To disconnect you must gently turn and pull the piece.*

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*The Dent-Flex's air motor coupling system to the contra angle, straight handpiece are in accordance with ISO 3964 norm.*

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1 - Turn the latch to the right. Insert the bur gently turning it, lock the bur turning the latch to the left. Pull the bur to check if it's really fixed.

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
1 - Press the central part of the activator button, located on the superior part of the Contra Angle's head to open the clamp in order to enable the bur's fixing and removal (Fig. 5)

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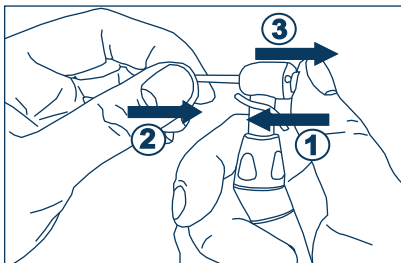*Using only 1 bits according to ISO 1797-1 Type 1, the following specifications: Ø 2.334 mm to 2.35 mm ; lenght minimum of 9 mm and maximum 12 mm ; lenght minimum of 22 mm and a maximum of 28 mm drill.*

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*1 - To remove or insert the drill, loosen the inner clamp by turning the knurled sleeve to the right (Fig. 6) , from the closed position to the open position.*

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*2 - Fit the bur into the chuck until it touches the bottom (Fig. 7).*

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
*3 - Soon after, you should turn the locking sleeve to the left until it reaches its threshold, thus realizing the drill fixing (Fig. 8), from the open position to the closed position.*

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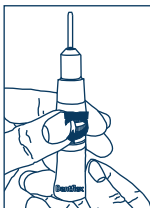*Avoid closing the straight handpiece's clamp without the drill or rest pin, this procedure may damage it.*

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*Using only 1 bits according to the standard ISO 1797-1 type 2, with the following specifications: Ø 2.334 mm to 2.35 mm; length minimum of 15 mm and maximum 30 mm; length minimum 30 mm and maximum 45 mm drill.*

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*1 - Lubricate before each sterilization cycle.*

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*2 - To lubricate, keep the oil bottle upright, insert your applicator at the air motor's air entering hole and then inject oil for 2 seconds.*

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*The contra angle and straight hand piece lubrication must be done through oil injection on its base, you must keep it for 2 seconds*

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*The Dentflex lubricant spray or other specific lubricant for dental handpieces should be used.*

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Fig. 13 - White Mineral Lubricator Oil for Dental Equipment.

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- Do not expose to temperature above 55°C.
- Keep the bottle far from spark or fire.
- Do not pierce or burn the bottle neither if it's empty.
- Avoid contact to the eyes.

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*Like any other electromedical device, Low Speed is composed of several materials, such as metals, rubbers, etc. Therefore, in order to avoid environmental contamination or the improper use of the equipment after its destruction, it must be disposed of in an appropriate place (following local legislation for waste disposal conditions).*

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*Dent-Flex Ind. E Com. Ltda: It declares that the materials used in its Applied Parts (according to the NBR IEC 60601-1 standard) in Low Speed have been widely used in the dental field over the years, guaranteeing its biocompatibility. In addition, the consumable items of the equipment are: lubricating oil, distilled water and drills.*

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*All recommendations for use are found in this Manual, but if any problems or malfunctions are detected please contact the Dentflex Technical Support Center to correct the problem. Do not attempt to open the product and / or repair it yourself or with help of persons not trained / authorized to do so, this may aggravate the problem or even generate faults that could compromise product safety.*

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SENSITIVITY TO PREDICTABLE ENVIRONMENTAL CONDITIONS NORMAL USE SITUATIONS

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*The Low Speed was designed to have a maximum noise level of 80 dB, as long as it is installed, kept clean and maintained, transported and operated according to these use instructions.*

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*For warranty's information see the certificate of the product. The warranty's product covers manufacture defects counted from the date of sale to the first client or the delivery of the warranty certificate. The warranty becomes invalid in case any of the following situations is verified:*

- *Installation and/or operation in disagreement with this manual;*
- *Fall;*
- *Damages to the external finishing and/ or accessories whose the manufacturer has not given any cause;*
- *Without lubrication.*

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*With the purpose of offering a service that guarantees 100% satisfaction, we present to the customer the Dentflex Technical Support Center, an innovation in the service that came to provide agility, safety and transparency in the maintenance process of your Dentflex product. We have an infrastructure and a qualified professionals' team, guaranteeing the most complete technical assistance for the entire Dentflex product line.*

- Trained technicians and engineers;
- Last generation machinery and equipment;
- Original components.

*To use the service offered by the Dentflex Technical Support Center, please, contact us: E-mail: [sac@dentflex.com.br](mailto:sac@dentflex.com.br) / Phones: + 55 (16) 9 9404-4435 and + 55 (16) 2133-0000.*

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