Dentflex

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This product is a Multiplier Contra Angle, intended for applications where it is necessary to unite torque and extreme precision at the drill's active tip. It is coupled to the air motor by a quick coupling system, and has a 1:5 multiplier rotation system.

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The Contra Angle Prime M 1:5 is used for the dentistry and prosthesis' areas; for the work of refinement and completion in the preparation of aesthetic prosthesis, inlay, onlay, smoothing, cavity preparations and facets.

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It should be used by a qualified user observing the regulations in force for preventing accidents and safety at work, as well as these instructions for use. Its use for any other purpose is not allowed. Handling it improperly constitutes a danger source.

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The equipment can be disinfected with alcohol 70° GL, applied on its surface with a cloth or spray.

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Do not clean the equipment using ultra sonic cleaners

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The equipment must be sterilized before and after each treatment.

Pack the product in packaging for sterilization and dispose only in an autoclave at a maximum temperature 135° C and 2.2 Bar.

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Sterilize the equipment before using it for the first time. Use compressed dry air, filtered and with no oil.

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*Keep in a place protected from the sun, heat, dust and humidity;
Keep away from chemicals and cleaning agents;
Carry out the transport only in its respective packaging;
Ambient temperature (transport and storage): -10°C to 55°C;
Relative air humidity (transport and storage): 0% to 85%;
Atmospheric pressure: 70kPa to 106kPa;
Maximum stacking: 8 units.*

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1 - Press the Contra Angle over the Air Motor coupling until it gets fixed. To disconnect you must gently turn and pull the piece.

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2 - Test the contra-angle attachment to the micromotor before each use. Lubricate and disinfect/before using the handpiece for the first time and each time it is used, sterilize it.

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The Dent-Flex's air motor coupling system to the contra angle is in accordance with ISO 3964 norm.

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3 – If the air motor has a lighting system, after fitting the contra angle on it, rotate the handpiece until the positioning tab engages the micromotor.

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Fig. 2. Lighting system air motor coupling.

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It does not provide its own light, it only conducts the light produced by the air motor that has a lighting system; Contra Angle with fiber optic: used in air motors with lighting system; Contra Angle without fiber optic: used in air motors with e-type system.

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1 – Press the central part of the activator button, located on the superior part of the Contra Angle's head to open the clamp in order to enable the bur's fixing and removal (Fig. 3)

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Using only burs according to ISO 1797-1 Type 1, with the following specifications: Ø 2.334 mm to 2.35 mm ; minimum stem lenght of 9 mm and maximum of 12 mm; minimum bur lenght of 22 mm and maximum of 28 mm.

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1 - Lubricate before each sterilization cycle.

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2 - To lubricate, keep the oil bottle upright, insert your applicator at the air motor's air entering hole and then inject oil for 2 seconds.

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The Dentflex lubricant spray or other specific lubricant for dental handpieces should be used.

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The straight hand piece lubrication must be done through oil injection on its base, you must keep it for 2 seconds

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Fig. 5 - White Mineral Lubricator Oil for Dental Equipment.

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- Do not expose to temperature above 55°C.
- Keep the bottle far from spark or fire.
- Do not pierce or burn the bottle neither if it's empty.
- Avoid contact to the eyes.

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Like any other electromedical device, the Contra Angle is composed of several materials, such as metals, rubbers, etc. Therefore, in order to avoid environmental contamination or the improper use of the equipment after its destruction, it must be disposed of in an appropriate place (following local legislation for waste disposal conditions).

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Dent-Flex Ind. e Com. Ltda: It declares that the materials used in its Applied Parts (according to the NBR IEC 60601-1 standard) in High Speed have been widely used in the dental field over the years, guaranteeing its biocompatibility. In addition, the consumable items of the equipment are: lubricating oil, distilled water and drills.

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All recommendations for use are found in this Manual, but if any problems or malfunctions are detected please contact the Dentflex Technical Support Center to correct the problem. Do not attempt to open the product and / or repair it yourself or with help of persons not trained / authorized to do so, this may aggravate the problem or even generate faults that could compromise product safety.

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SENSITIVITY TO PREDICTABLE ENVIRONMENTAL CONDITIONS NORMAL USE SITUATIONS

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The Contra Angle was designed to have a maximum noise level of 80 dB, as long as it is installed, kept clean and maintained, transported and operated according to these use instructions.

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For warranty's information see the certificate of the product. The warranty's product covers manufacture defects counted from the date of sale to the first client or the delivery of the warranty certificate. The warranty becomes invalid in case any of the following situations is verified:

- *Installation and/or operation in disagreement with this manual;*
- *Fall;*
- *Damages to the external finishing and/ or accessories whose the manufacturer has not given any cause;*
- *Without lubrication.*

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E-mail: sac@dentflex.com.br / Telefones (16) 99404-4435 e (16) 2133-0000.

With the purpose of offering a service that guarantees 100% satisfaction, we present to the customer the Dentflex Technical Support Center, an innovation in the service that came to provide agility, safety and transparency in the maintenance process of your Dentflex product. We have an infrastructure and a qualified professionals' team, guaranteeing the most complete technical assistance for the entire Dentflex product line.

- Trained technicians and engineers;
- Last generation machinery and equipment;
- Original components.

To use the service offered by the Dentflex Technical Support Center, please, contact us: E-mail: sac@dentflex.com.br / Phones: + 55 (16) 9 9404-4435 and + 55 (16) 2133-0000.

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