Dentflex

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The low speed is a transmission equipment, when coupled to the air motor has the aim to fix and rotate burs used in many dental procedures.

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These devices are used for bone drilling, fastening screws, orthognathic surgery, maxilo-facial surgeries and several other dental or medical surgical procedures. It has a fundamental role in the oral rehabilitation of the patient and their dental health.

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The Low Speed is designated for dental procedures and can only be used and installed for a qualified and capacitated professional on dental area. Its use for any other purpose is not allowed. Handling it improperly constitutes a danger source.

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The equipment can be disinfected with alcohol 70° GL, applied on its surface with a cloth or spray.

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Do not clean the equipment using ultra sonic cleaners

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The equipment must be sterilized before and after each treatment.

Pack the product in packaging for sterilization and dispose only in an autoclave at a maximum temperature 135° C and 2.2 Bar.

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Sterilize the equipment before using it for the first time. Use compressed dry air, filtered and with no oil.

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*Keep in a place protected from the sun, heat, dust and humidity;
Keep away from chemicals and cleaning agents;
Carry out the transport only in its respective packaging;
Ambient temperature (transport and storage): -10°C to 55°C;
Relative air humidity (transport and storage): 0% to 85%;
Atmospheric pressure: 70kPa to 106kPa;
Maximum stacking: 8 units.*

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1 - Press the Contra Angle over the Air Motor coupling until it gets fixed. To disconnect you must gently turn and pull the piece.

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The Dent-Flex's air motor coupling system to the contra angle is in accordance with ISO 3964 norm.

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1 – Turn the latch to the right. Insert the bur gently turning it, lock the bur turning the latch to the left. Pull the bur to check if it's really fixed.

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1 – Press the central part of the activator button, located on the superior part of the Contra Angle's head to open the clamp in order to enable the bur's fixing and removal (Fig. 3)

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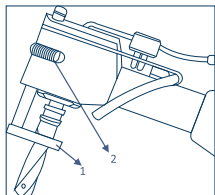
Using only burs according to ISO 1797-1 Type 1, with the following specifications: Ø 2.334 mm to 2.35 mm ; minimum stem length of 9 mm and maximum of 12 mm; minimum bur length of 22 mm and maximum of 28 mm.

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
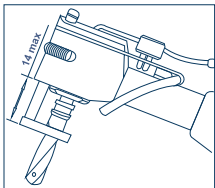
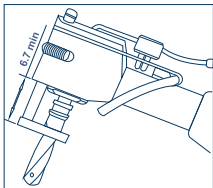
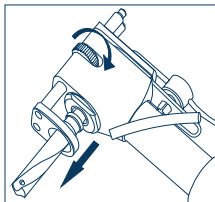
To adjust the position of the limiter simply turn the adjusting nut position.

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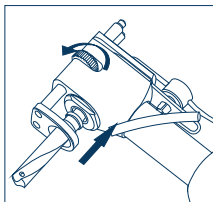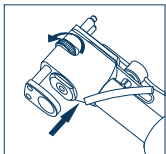
2 - Fit the set limiter by aligning the hole nut with the threaded shaft until they snap into the same place.

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3 - Turn the nut continuously in the direction of the arrow while pushing the set limiter toward the nut.

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4 - When the set limiter start moving due to the nut spin, the process will be finished.

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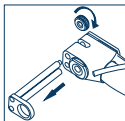
1 - Turn the nut continuously in the direction of the arrow until the limiter set stop moving.

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2 - Pull the limiter set out of the contra angle head, then move the nut out of the way.

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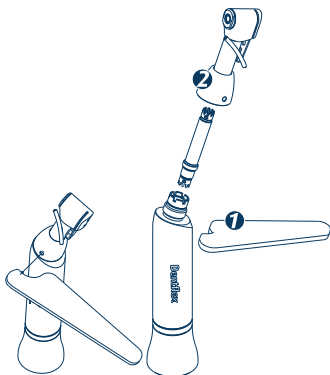
With wrench # 1 unscrew contra angle's nut # 2 , in a way to separate the head and the front transmission set from the equipment's body. Then execute the asepsis and fit up the equipment using wrench # 1.

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1 - Lubricate before each sterilization cycle.

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2 - To lubricate, keep the oil bottle upright, insert your applicator at the air motor's air entering hole and then inject oil for 2 seconds.

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The Dentflex lubricant spray or other specific lubricant for dental handpieces should be used.

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The straight hand piece lubrication must be done through oil injection on its base, you must keep it for 2 seconds

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Fig. 5 - White Mineral Lubricator Oil for Dental Equipment.

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- *Do not expose to temperature above 55°C.*
- *Keep the bottle far from spark or fire.*
- *Do not pierce or burn the bottle neither if it's empty.*
- *Avoid contact to the eyes.*

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Like any other electromedical device, the Contra Angle is composed of several materials, such as metals, rubbers, etc. Therefore, in order to avoid environmental contamination or the improper use of the equipment after its destruction, it must be disposed of in an appropriate place (following local legislation for waste disposal conditions).

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Dent-Flex Ind. e Com. Ltda: It declares that the materials used in its Applied Parts (according to the NBR IEC 60601-1 standard) in High Speed have been widely used in the dental field over the years, guaranteeing its biocompatibility. In addition, the consumable items of the equipment are: lubricating oil, distilled water and drills.

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All recommendations for use are found in this Manual, but if any problems or malfunctions are detected please contact the Dentflex Technical Support Center to correct the problem. Do not attempt to open the product and / or repair it yourself or with help of persons not trained / authorized to do so, this may aggravate the problem or even generate faults that could compromise product safety.

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SENSITIVITY TO PREDICTABLE ENVIRONMENTAL CONDITIONS NORMAL USE SITUATIONS

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The Contra Angle was designed to have a maximum noise level of 80 dB, as long as it is installed, kept clean and maintained, transported and operated according to these use instructions.

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For warranty's information see the certificate of the product. The warranty's product covers manufacture defects counted from the date of sale to the first client or the delivery of the warranty certificate. The warranty becomes invalid in case any of the following situations is verified:

- Installation and/or operation in disagreement with this manual;*
- Fall;*
- Damages to the external finishing and/ or accessories whose the manufacturer has not given any cause;*
- Without lubrication.*

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E-mail: sac@dentflex.com.br / Telefones (16) 99404-4435 e (16) 2133-0000.

With the purpose of offering a service that guarantees 100% satisfaction, we present to the customer the Dentflex Technical Support Center, an innovation in the service that came to provide agility, safety and transparency in the maintenance process of your Dentflex product. We have an infrastructure and a qualified professionals' team, guaranteeing the most complete technical assistance for the entire Dentflex product line.

- Trained technicians and engineers;
- Last generation machinery and equipment;
- Original components.

To use the service offered by the Dentflex Technical Support Center, please, contact us: E-mail: sac@dentflex.com.br / Phones: + 55 (16) 9 9404-4435 and + 55 (16) 2133-0000.

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