# Dentflex

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*The Bicarbonate Jet is used in dental medicine for dental prophylaxis. It works connected to the dental unit, with pneumatic mechanical system obtained by issuing jet of crystals of sodium bicarbonate, driven by air and water pressure.*

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*The Bicarbonate Jet removes dark stains from teeth caused by smoking, coffee, tea, etc., associated with dental plaque and not the calculation.*

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*The Bicarbonate Jet is designated for dental procedures and can only be used and installed by a qualified and capacited professional on dental area. Its use for any other purpose is not allowed. Handling it improperly constitutes a danger source.*

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*The equipment can be disinfected with alcohol 70° GL, applied on its surface with a cloth or spray.*

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*Do not clean the equipment using ultra sonic cleaners*

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*Use compressed dry air, filtered and with no oil.*

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**Empilhamento máximo: 8 unidades.**

*Keep the equipment in you original packaging;*

*Keep in a place protected from the sun, heat, dust and humidity;*

*Keep away from chemicals and cleaning agents;*

*Avoid weight on the equipment;*

*Empty the reservoir of baking powder while equipment is inoperative;*

*Carry out the transport only in its respective packaging;*

*Ambient temperature (transport and storage): -10°C to 55°C;*

*Relative air humidity (transport and storage): 0% to 85%;*

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Before installing your equipment check:

- If your equipment has a connection type borden (2 holes);
- If the water and air are with the pressure measured (see technical data);
- If there is powder of sodium bicarbonate in the reservoir.

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1 - Connect the nozzle (sterile) in the body of the equipment.

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2 - Connect the Bicarbonate Jet to the borden terminal of the unit (Fig. 1).

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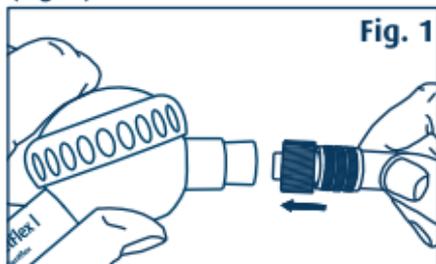
Fig. 1

3 - Ajuste a saída de água do equipo.

3 - Adjust the water output from to the dental unit.

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5 - Fill the container with bicarbonate powder until the internal step of the reservoir. (indicated on the inside of the reservoir).

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*7 - Drive and control intensity of Bicarbonate Jet are conducted through the unit pedal.*

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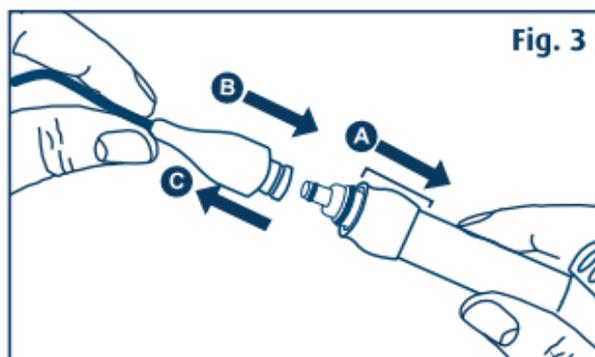
*8 - To engage the bicarbonate jet nozzle, pull the device from the body of the jet in the direction indicated (A), then pull the tip to direction (B). (Fig. 3)*

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*9 - To disengage the nozzle from the bicarbonate jet, pull the device from the body of the jet in the direction indicated (A), then remove the tip to direction (C). (Fig. 3)*

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*1 - To get the best result of bicarbonate jet, we recommend that respects the distance of the handpiece in relation to the tooth (5mm) with an inclination of 30° to 45 ° describing small circular motions on the teeth.*

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*2 - Use the sucker in the patient during the procedure.*

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*3 - To avoid unpleasant sensations in patients, it should be directed to the bicarbonate jet occlusal edge and not to the gingival sulcus.*

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*4 - Make sure that the Bicarbonate Jet is completely stopped before removing it from the mouth of the patient.*

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*5 - Instruct the patient not to smoke and / or drink and eat food with coloring two hours after the procedure, because the Bicarbonate Jet removes the protein layer covering the surface of the enamel, and will take about this time to regroup.*

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*1 - This device is contraindicated for use in patients who have serious respiratory problems or kidney who undergo hemodialysis, these cases should have medical care.*

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*2 - Do not drive the device against mucosal tissue (gum). It can cause bleeding gums.*

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*3 - Ensure the proper functioning of the water outlet. The flaw in the system can cause clogging of the equipment.*

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*4 - Eye injury can be caused if the equipment is driven toward the unprotected eyes.*

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*5 - We recommend the use of mask and goggles for the application of Bicarbonate Jet.*

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*6 - Do not remove the reservoir cap while the machine is under pressure.*

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## 1 - Nozzle of the Needle and Equipment:

- Disconnect the equipment from the equipment.
- Remove the nozzle and wash with 70% alcohol.
- After disinfection, sterilize in an autoclave at 135 °C for 20 minutes.

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## 2 - Reservoir Body and Cover:

- Remove all bicarbonate residue from the body container of the equipment.
- Wash with water and disinfect with 70% alcohol.

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Only the equipment's nozzle and needle should be sterilized in autoclave.

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*This procedure should be done immediately after using the appliance. Follow the step-by-step below to avoid clogging the hole through which the sodium bicarbonate.*

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**1 - Desacople o corpo do equipamento do equipo.**

*1 - Disconnect the body from the equipment of the unit.*

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*2 - Discard any remaining sodium bicarbonate in the reservoir.*

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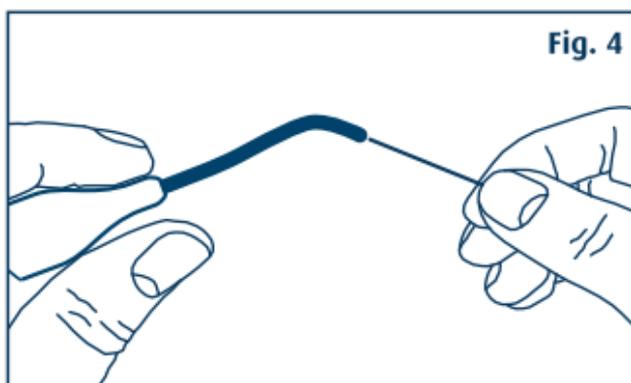
*3 - Remove the nozzle of the equipment and apply a jet of air to the holes in the body and tip..*

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*4 - Insert the needle that comes with the product into the holes in the body and nozzle as shown. (Fig. 4)*

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5 - Reapply a jet of air to remove any dust residue that still remains inside the hole.

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6 - Use only the needle that came with the original equipment.

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Types of recurring obstructions if the cleaning procedure is not performed as instructed:

• **Powder obstruction:** caused by the bicarbonate's own powder that turns into a crystalline stone and makes it impossible to clear the nozzle;

• **Water obstruction:** caused by the particles contained in the non-distilled water that form organic sediment preventing the water spray.

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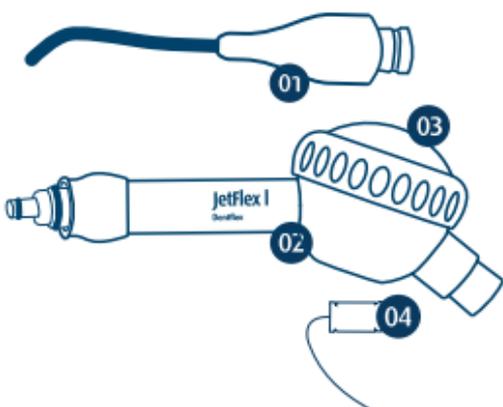
04 - Agulha para limpeza da parte frontal do jato

04 - Needle for cleaning the front of the jet

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*Like any other electromedical device, Jetflex I is composed by several materials, such as metals, rubbers, etc. Therefore, in order to avoid environmental contamination or the improper use of the equipment after its destruction, it must be disposed of in an appropriate place (following local legislation for waste disposal conditions).*

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*Dent-Flex Ind. E Com. Ltda: It declares that the materials used in its Applied Parts (according to the NBR IEC 60601-1 standard) in the Jetflex I have been widely used in the dental field over the years, guaranteeing its biocompatibility. In addition, the consumable items of the equipment are: lubricating oil, distilled water and drills.*

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*All recommendations for use are found in this Manual, but if any problems or malfunctions are detected please contact the Dentflex Technical Support Center to correct the problem. Do not attempt to open the product and / or repair it yourself or with help of persons not trained / authorized to do so, this may aggravate the problem or even generate faults that could compromise product safety.*

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SENSITIVITY TO PREDICTABLE ENVIRONMENTAL CONDITIONS NORMAL USE SITUATIONS

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*The Bicarbonate Jet was designed to have a maximum noise level of 80 dB, as long as it is installed, kept clean and maintained, transported and operated according to these use instructions.*

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*For warranty's information see the certificate of the product. The warranty's product covers manufacture defects counted from the date of sale to the first client or the delivery of the warranty certificate. The warranty becomes invalid in case any of the following situations is verified:*

- Installation and/or operation in disagreement with this manual;*
- Fall;*
- Damages to the external finishing and/ or accessories whose the manufacturer has not given any cause.*

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*With the purpose of offering a service that guarantees 100% satisfaction, we present to the customer the Dentflex Technical Support Center, an innovation in the service that came to provide agility, safety and transparency in the maintenance process of your Dentflex product. We have an infrastructure and a qualified professionals' team, guaranteeing the most complete technical assistance for the entire Dentflex product line.*

- Trained technicians and engineers;
- Last generation machinery and equipment;
- Original components.

*To use the service offered by the Dentflex Technical Support Center, please, contact us: E-mail: [sac@dentflex.com.br](mailto:sac@dentflex.com.br) / Phones: + 55 (16) 9 9404-4435 and + 55 (16) 2133-0000.*

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