Dentflex

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KIT INTRA SIGMA AIR PRIME

Intra Sigma Air Prime Kit • Kit Intra Sigma Air Prime

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The High Speed is one dental unit accessory. It is easy to handle this equipment, it may be coupled in any kind of unit, by following all the products directions for its adequate use.

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The equipment is used in dental treatments and for the confection of dental prothesis.

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The High Speed Sigma Air is designated for dental procedures and can only be used and installed by a qualified and capacitated professional on dental area. Its use for any other purpose is not allowed. Handling it improperly constitutes a danger source.

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1 - Fit the high speed to the unit's terminal observing the ducts direction.

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2 - Screw the unit's terminal extremity to get fixed to the equipment.

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The dental unit's terminal must be manufactured in accordance with ISO 9168 norm.

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1 – Fit the bur manually into the clamp and press against the metallic support of the bur extractor. (Fig. 3).

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2 – To remove the bur, insert the equipment head on latch pin side, and then hold the bur and press the head on the extractor (Fig. 4).

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1 – Press the central part of the activator button, located on the superior part of the high speed head to open the clamp in order to enable the bur's fixing (figure 5) and removal (figure 6).

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Use only burs according to norm ISO 1797-1 type 3 with the following dimensions:

a - diameter between 1.59 and 1.60mm

b - minimum recommended length of the rotary instrument shaft is 12mm.

c - total maximum recommended length of the rotary tool is 22mm.

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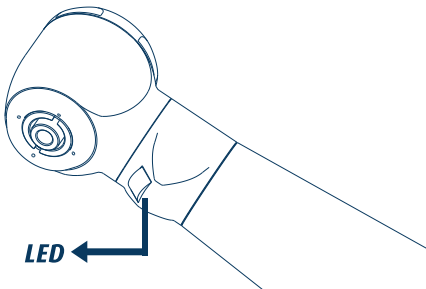
The luminosity of the LED can vary according to the working pressure and rotation of the turbine.

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1 - *Lubricate before each sterilization cycle.*

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2 - *To lubricate, keep the oil bottle upright, insert your applicator at the high speed entry and inject oil for 2 seconds.*

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3 - *After lubrication the high speed must be autoclaved.*

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The Dentflex lubricant spray¹⁰ or other specific lubricant for dental handpieces should be used.

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Fig. 9 - White Mineral Lubricator Oil for Dental Equipment.

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- Do not expose to temperature above 55°C.
- Keep the bottle far from spark or fire.
- Do not pierce or burn the bottle neither if it's empty.
- Avoid contact to the eyes.

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The low speed is a transmission equipment, when coupled to the air motor has the aim to fix and rotate burs used in many dental procedures.

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The Low Speed is designated for dental procedures and can only be used and installed for a qualified and capacitated professional on dental area. Its use for any other purpose is not allowed. Handling it improperly constitutes a danger source.

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1 - Fit the air motor to the unit's terminal observing the ducts direction (Fig. 1). Screw the unit's terminal extremity to get fixed to the equipment (Fig. 2).

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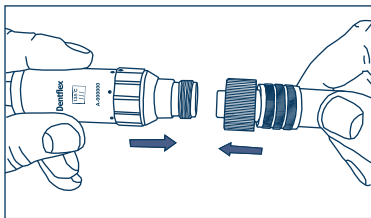
Fig. 1 - Encaixe do Micromotor ao Terminal do Equipó.

Fig. 1 - Fit the Air Motor to the Unit's Terminal.

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The dental unit's terminal must be manufactured in accordance with ISO-9168 norm, type A connector

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1 – Press the Contra Angle over the Air Motor coupling until it gets fixed. To disconnect you must gently turn and pull the piece.

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The Dent-Flex's air motor coupling system to the contra angle, straight handpiece are in accordance with ISO 3964 norm.

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The Micromotor MI 01 Prime RI has an internal cooling system. Attach to the Micromotor the Contra Angle or Straight Piece that is compatible with this connection. The hose must be connected from the base of the contra-angle or straight piece until to the top.

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Fig. 4 - Micromotor's with an internal cooling system.

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Fig. 5 - Micromotor's fit with an internal cooling system to the Contra-Angle and Straight Handpiece.

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The Micromotor MI 01 Prime RE has an external cooling system. Attach to the Micromotor the Contra Angle or Straight Piece that is compatible with this connection. The hose must be connected from the base of the contra-angle or straight piece until to the top.

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Fig. 6 - Micromotor's with an external cooling system.

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Fig. 7 - Micromotor's fit with an external cooling system to the Contra-Angle and Straight Handpiece.

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1 – Turn the latch to the right. Insert the bur gently turning it, lock the bur turning the latch to the left. Pull the bur to check if it's really fixed.

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1 – Press the central part of the activator button, located on the superior part of the Contra Ângulo's head to open the clamp in order to enable the bur's fixing and removal (Fig. 5)

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Using only 1 bits according to ISO 1797-1 Type 1, the following specifications: Ø 2.334 mm to 2.35 mm ; lenght minimum of 9 mm and maximum 12 mm ; lenght minimum of 22 mm and a maximum of 28 mm drill.

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The kit with 3 pieces does not come with a straight handpiece.

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1 - To remove or insert the drill, loosen the inner clamp by turning the knurled sleeve to the right (Fig. 6) , from the closed position to the open position.

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2 - Fit the bur into the chuck until it touches the bottom (Fig. 7).

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3 - Soon after, you should turn the locking sleeve to the left until it reaches its threshold, thus realizing the drill fixing (Fig. 8), from the open position to the closed position.

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Avoid closing the straight handpiece's clamp without the drill or rest pin, this procedure may damage it.

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Using only 1 bits according to the standard ISO 1797-1 type 2, with the following specifications: Ø 2.334 mm to 2.35 mm; length minimum of 15 mm and maximum 30 mm; length minimum 30 mm and maximum 45 mm drill.

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1 - Lubricate before each sterilization cycle.

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2 - To lubricate, keep the oil bottle upright, insert your applicator at the air motor's air entering hole and then inject oil for 2 seconds.

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Fig. 8 - Midwest Connection Lubrication.

Fig. 8 - Lubricación Conexión Midwest.

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Fig. 9 - Borden Connection Lubrication.

Fig. 9 - Lubricación Conexión Borden.

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Fig. 10 - Contra Angle Lubrication.

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Fig. 11 - Straight Hand Piece Lubrication.

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The contra angle and straight hand piece lubrication must be done through oil injection on its base, you must keep it for 2 seconds

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The Dentflex lubricant spray or other specific lubricant for dental handpieces should be used.

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Fig. 12 - White Mineral Lubricator Oil for Dental Equipment.

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The equipment can be disinfected with alcohol 70° GL, applied on its surface with a cloth or spray.

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Do not clean the equipment using ultra sonic cleaners

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The equipment must be sterilized before and after each treatment.

Pack the product in packaging for sterilization and dispose only in an autoclave at a maximum temperature 135° C and 2.2 Bar.

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Sterilize the equipment before using it for the first time. Use compressed dry air, filtered and with no oil.

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Keep in a place protected from the sun, heat, dust and humidity;

Keep away from chemicals and cleaning agents;

Carry out the transport only in its respective packaging;

Ambient temperature (transport and storage): -10°C to 55°C;

Relative air humidity (transport and storage): 0% to 85%;

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Like any other electromedical device, the equipment is composed of several materials, such as metals, rubbers, etc. Therefore, in order to avoid environmental contamination or the improper use of the equipment after its destruction, it must be disposed of in an appropriate place (following local legislation for waste disposal conditions).

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Dent-Flex Ind. E Com. Ltda: It declares that the materials used in its Applied Parts (according to the NBR IEC 60601-1 standard) in the equipment have been widely used in the dental field over the years, guaranteeing its biocompatibility. In addition, the consumable items of the equipment are: lubricating oil, distilled water and drills.

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All recommendations for use are found in this Manual, but if any problems or malfunctions are detected please contact the Dentflex Technical Support Center to correct the problem. Do not attempt to open the product and / or repair it yourself or with help of persons not trained / authorized to do so, this may aggravate the problem or even generate faults that could compromise product safety.

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SENSITIVITY TO PREDICTABLE ENVIRONMENTAL CONDITIONS NORMAL USE SITUATIONS

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The equipment was designed to have a maximum noise level of 80 dB, as long as it is installed, kept clean and maintained, transported and operated according to these use instructions.

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For warranty's information see the certificate of the product. The warranty's product covers manufacture defects counted from the date of sale to the first client or the delivery of the warranty certificate. The warranty becomes invalid in case any of the following situations is verified:

- Installation and/or operation in disagreement with this manual;*
- Fall;*
- Damages to the external finishing and/ or accessories whose the manufacturer has not given any cause;*
- Without lubrication.*

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E-mail: sac@dentflex.com.br / Telefones (16) 99404-4435 e (16) 2133-0000.

With the purpose of offering a service that guarantees 100% satisfaction, we present to the customer the Dentflex Technical Support Center, an innovation in the service that came to provide agility, safety and transparency in the maintenance process of your Dentflex product. We have an infrastructure and a qualified professionals' team, guaranteeing the most complete technical assistance for the entire Dentflex product line.

- Trained technicians and engineers;
- Last generation machinery and equipment;
- Original components.

To use the service offered by the Dentflex Technical Support Center, please, contact us: E-mail: sac@dentflex.com.br / Phones: + 55 (16) 9 9404-4435 and + 55 (16) 2133-0000.

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