Dentflex

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The micro saws Dentflex are mechanical devices when connected to an electric or pneumatic motor transform its rotary motion into reciprocating and oscillating movement, causing the blade at this tip connected execute and wear bone cuts with great precision.

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The equipment is used in surgical procedures for facial restoration, bone reconstruction, orthognathic surgery, correction of traumatic deformities and facial clefts.

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The The Micro Saw is designated for dental procedures and can only be used and installed for a qualified and capacitated professional on dental area. Its use for any other purpose is not allowed. Handling it improperly constitutes a danger source.

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The equipment can be disinfected with alcohol 70° GL, applied on its surface with a cloth or spray.

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Do not clean the equipment using ultra sonic cleaners

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The equipment must be sterilized before and after each treatment. Pack the product in packaging for sterilization and dispose only in an autoclave at a maximum temperature 135° C and 2.2 Bar.

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Sterilize the equipment before using it for the first time. Use compressed dry air, filtered and with no oil.

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*Keep in a place protected from the sun, heat, dust and humidity;
Keep away from chemicals and cleaning agents;
Carry out the transport only in its respective packaging;
Ambient temperature (transport and storage): -10°C to 55°C;
Relative air humidity (transport and storage): 0% to 85%;
Atmospheric pressure: 70kPa to 106kPa;
Maximum stacking: 8 units.*

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1 - Press the handpiece over the Air Motor coupling until it gets fixed. To disconnect you must gently turn and pull the piece.

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The Dent-Flex's air motor coupling system to the he Straight and Angular Surgery Piece Dentflex is in accordance with ISO 3964 norm.

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
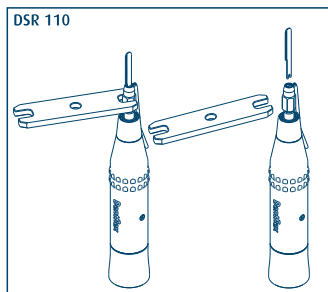
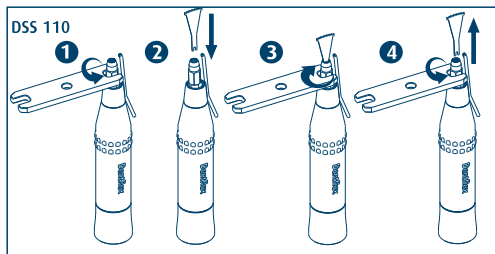
- 1 - Loosen the lock nut counter-clockwise.
- 2 - Correctly place the saw on the bottom of the stem.
- 3 - Tighten the nut with the key clockwise.
- 4 - To remove the saw rotate the nut with the key in counter-clockwise.

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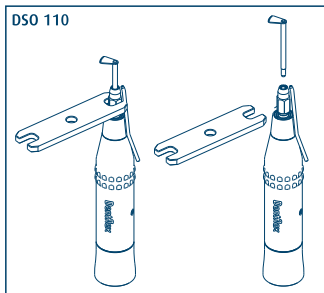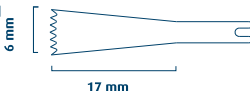
The Micro Saw dentflex can be used with any of the domestic market with the blade's specifications below:

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
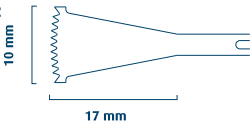
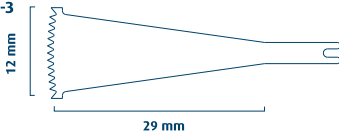
The DSS 110 Micro Saw accompanies a SDS-1 sagittal blade model. The other models are optional.

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







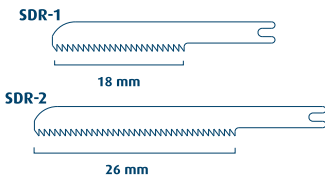
The DSR 110 Micro Saw accompanies a SDR-1 reciprocating blade model. The SDR-2 model is optional.

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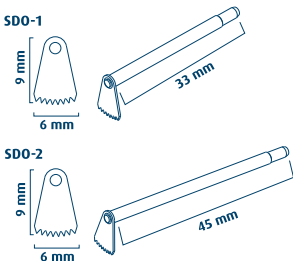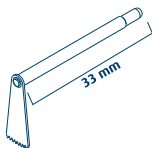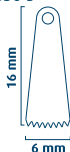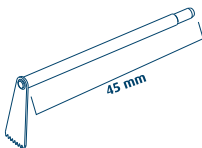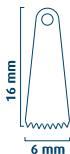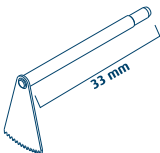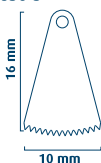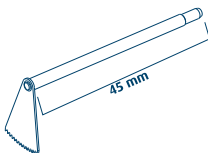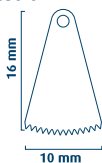The DSR 110 Micro Saw accompanies a SDO-3 oscillating blade model. The other models are optional.

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1 - Lubricate before each sterilization cycle.

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2 - To lubricate, keep the oil bottle upright, insert your applicator at the air motor's air entering hole and then inject oil for 2 seconds.

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3 - After lubrication remove all the oil excess at the pneumatic micromotor for ± 15 seconds. After removing the oil excess the micromotor must be autoclaved.

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The Dentflex lubricant spray or other specific lubricant for dental handpieces should be used.

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Fig. 10 - White Mineral Lubricator Oil for Dental Equipment.

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- Do not expose to temperature above 55°C.
- Keep the bottle far from spark or fire.
- Do not pierce or burn the bottle neither if it's empty.
- Avoid contact to the eyes.

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Like any other electromedical device, the Micro Saw is composed of several materials, such as metals, rubbers, etc. Therefore, in order to avoid environmental contamination or the improper use of the equipment after its destruction, it must be disposed of in an appropriate place (following local legislation for waste disposal conditions).

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Dent-Flex Ind. E Com. Ltda: It declares that the materials used in its Applied Parts (according to the NBR IEC 60601-1 standard) in Low Speed have been widely used in the dental field over the years, guaranteeing its biocompatibility. In addition, the consumable items of the equipment are: lubricating oil, distilled water and drills.

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All recommendations for use are found in this Manual, but if any problems or malfunctions are detected please contact the Dentflex Technical Support Center to correct the problem. Do not attempt to open the product and / or repair it yourself or with help of persons not trained / authorized to do so, this may aggravate the problem or even generate faults that could compromise product safety.

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SENSITIVITY TO PREDICTABLE ENVIRONMENTAL CONDITIONS NORMAL USE SITUATIONS

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The Micro Saw was designed to have a maximum noise level of 80 dB, as long as it is installed, kept clean and maintained, transported and operated according to these use instructions.

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For warranty's information see the certificate of the product. The warranty's product covers manufacture defects counted from the date of sale to the first client or the delivery of the warranty certificate. The warranty becomes invalid in case any of the following situations is verified:

- *Installation and/or operation in disagreement with this manual;*
- *Fall;*
- *Damages to the external finishing and/ or accessories whose the manufacturer has not given any cause;*
- *Without lubrication.*

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E-mail: sac@dentflex.com.br / Telefones (16) 99404-4435 e (16) 2133-0000.

With the purpose of offering a service that guarantees 100% satisfaction, we present to the customer the Dentflex Technical Support Center, an innovation in the service that came to provide agility, safety and transparency in the maintenance process of your Dentflex product. We have an infrastructure and a qualified professionals' team, guaranteeing the most complete technical assistance for the entire Dentflex product line.

- Trained technicians and engineers;
- Last generation machinery and equipment;
- Original components.

To use the service offered by the Dentflex Technical Support Center, please, contact us: E-mail: sac@dentflex.com.br / Phones: + 55 (16) 9 9404-4435 and + 55 (16) 2133-0000.

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