# Dentflex

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*The low speed is a transmission equipment, when coupled to the air motor has the aim to fix and rotate burs used in many dental procedures.*

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*Straight and Angular Surgery Piece is recommended for implant surgery, orthognathic surgery and TMJ surgery. It has got a very important aim on oral rehabilitation and on oral health, beyond saving time of a procedure that manually would take too long to get finished, it also provides such perfection that only the mechanical equipment would afford.*

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*The Low Speed is designated for dental procedures and can only be used and installed for a qualified and capacitated professional on dental area. Its use for any other purpose is not allowed. Handling it improperly constitutes a danger source.*

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- 1 - *Intermitent use Handpiece: 2 minutes of operation and 5 minute break.*
- 2 - *An insufficient amount of water in spray can lead the handpiece overheating.*
- 3 - *Never touch soft tissue with the handpiece's body or tip.*
- 4 - *Do not continue working if it overheats.*

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The equipment can be disinfected with alcohol 70° GL, applied on its surface with a cloth or spray.

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*Do not clean the equipment using ultra sonic cleaners*

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*The equipment must be sterilized before and after each treatment.  
Pack the product in packaging for sterilization and dispose only in an autoclave at a maximum temperature 135° C and 2.2 Bar.*

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*Sterilize the equipment before using it for the first time. Use compressed dry air, filtered and with no oil.*

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Empilhamento máximo: 8 unidades.

*Keep in a place protected from the sun, heat, dust and humidity;*

*Keep away from chemicals and cleaning agents;*

*Carry out the transport only in its respective packaging;*

*Ambient temperature (transport and storage): -10°C to 55°C;*

*Relative air humidity (transport and storage): 0% to 85%;*

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1 – Press the handpiece over the Air Motor coupling until it gets fixed. To disconnect you must gently turn and pull the piece.

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The Dent-Flex's air motor coupling system to the he Straight and Angular Surgery Piece Dentflex is in accordance with ISO 3964 norm.

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1 – To remove or to insert the bur you must loosen the internal chuck, by turning the locker sleeve to the right (Fig. 2), observe that the turn has got two steps, Firstly a gentle turn then you must grab it stronger.

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
2 – Fit the bur into the chuck until it touches the bottom (Fig. 3).

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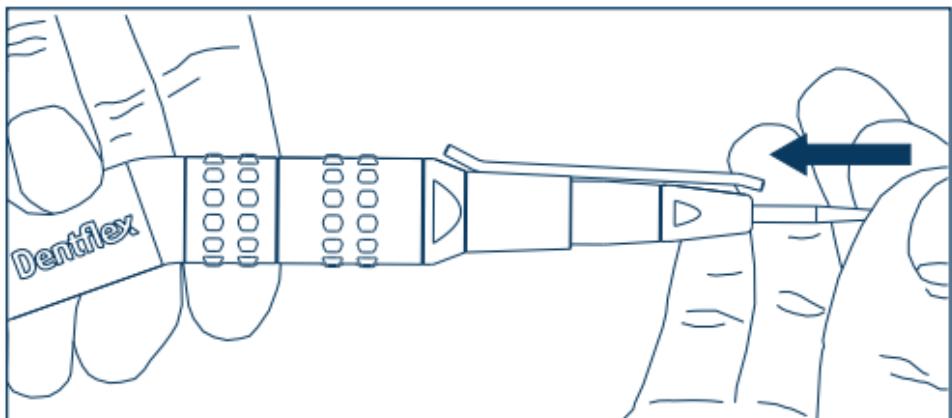
**3 – Soon after, you must turn the locker sleeve to the left until it gets to the initial point, so the bur will be fixed (Fig. 4).**

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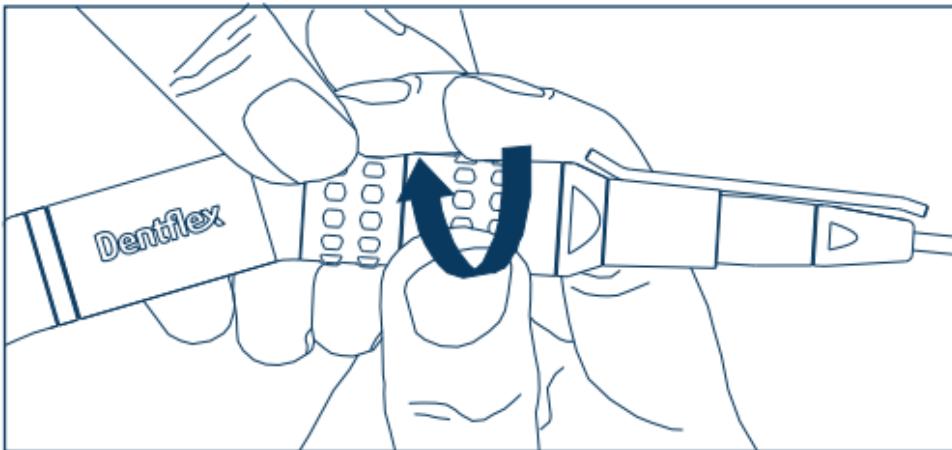
**4 - The Straight and Angular Surgery Piece Dentflex can be used with burs 45 and 70 mm in length. The 45mm bur needs a bearing part which can be inserted or removed according to the needs of use of burs.**

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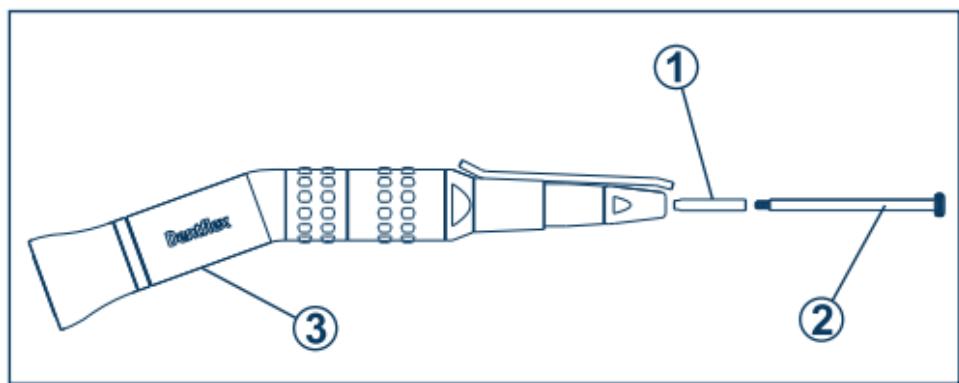**1- Support for bur**

**2- Puller support for bur**

**3 - Straight and Angular Surgery Piece**

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*5- To enter the part 1 (Support for bur) as shown in fig. 5, it is necessary to screw the part 2 (Puller Support for the bur) at the end of part 1 and insert it into the clamp until it snaps into the bottom of part 3 (Straight and Angular Surgery Piece) to finish the piece unscrew part 2 and remove thereof.*

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*6- If there is a need to use a 70mm bur part 1 (Support for bur) should be extracted, for this, screw the part 2 again (Puller Support for the bur) and then pull the thereof.*

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*Using only 1 bits according to the standard ISO 1797-1 type 2 , with the following specifications: Ø 2.334 mm to 2.35 mm ; lenght minimum of 15 mm and maximum 30 mm;lenght minimum 30 mm and maximum 45 mm drill.*

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*Never let the clamp closed without burs. After use, place the resting pin on the clamp, close it and store it.*

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
*1 - Lubricate before each sterilization cycle.*

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*2 - To lubricate, keep the oil bottle upright, insert your applicator at the air motor's air entering hole and then inject oil for 2 seconds.*

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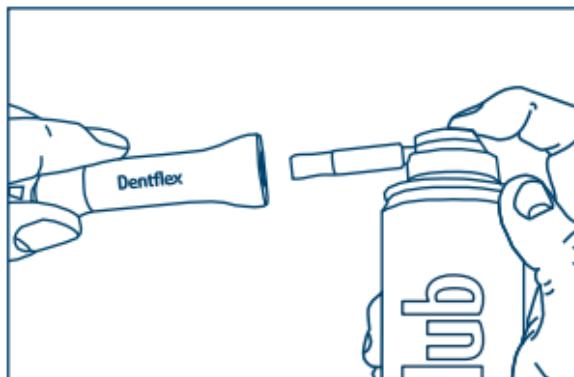*The Dentflex lubricant spray or other specific lubricant for dental handpieces should be used.*

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*The straight hand piece lubrication must be done through oil injection on its base, you must keep it for 2 seconds*

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*Fig. 7 - White Mineral Lubricator Oil for Dental Equipment.*

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- Do not expose to temperature above 55°C.
- Keep the bottle far from spark or fire.
- Do not pierce or burn the bottle neither if it's empty.
- Avoid contact to the eyes.

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*Like any other electromedical device, Low Speed is composed of several materials, such as metals, rubbers, etc. Therefore, in order to avoid environmental contamination or the improper use of the equipment after its destruction, it must be disposed of in an appropriate place (following local legislation for waste disposal conditions).*

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*Dent-Flex Ind. E Com. Ltda: It declares that the materials used in its Applied Parts (according to the NBR IEC 60601-1 standard) in Low Speed have been widely used in the dental field over the years, guaranteeing its biocompatibility. In addition, the consumable items of the equipment are: lubricating oil, distilled water and drills.*

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*All recommendations for use are found in this Manual, but if any problems or malfunctions are detected please contact the Dentflex Technical Support Center to correct the problem. Do not attempt to open the product and / or repair it yourself or with help of persons not trained / authorized to do so, this may aggravate the problem or even generate faults that could compromise product safety.*

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*SENSITIVITY TO PREDICTABLE ENVIRONMENTAL CONDITIONS NORMAL USE SITUATIONS*

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*The Low Speed was designed to have a maximum noise level of 80 dB, as long as it is installed, kept clean and maintained, transported and operated according to these use instructions.*

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*For warranty's information see the certificate of the product. The warranty's product covers manufacture defects counted from the date of sale to the first client or the delivery of the warranty certificate. The warranty becomes invalid in case any of the following situations is verified:*

- Installation and/or operation in disagreement with this manual;
- Fall;
- Damages to the external finishing and/ or accessories whose the manufacturer has not given any cause;
- Without lubrication.

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*With the purpose of offering a service that guarantees 100% satisfaction, we present to the customer the Dentflex Technical Support Center, an innovation in the service that came to provide agility, safety and transparency in the maintenance process of your Dentflex product. We have an infrastructure and a qualified professionals' team, guaranteeing the most complete technical assistance for the entire Dentflex product line.*

- Trained technicians and engineers;
- Last generation machinery and equipment;
- Original components.

*To use the service offered by the Dentflex Technical Support Center, please, contact us: E-mail: [sac@dentflex.com.br](mailto:sac@dentflex.com.br) / Phones: + 55 (16) 9 9404-4435 and + 55 (16) 2133-0000.*

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