Dentflex

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The Triple Syringe is an accessory of the dental equipment. Because it is an easy-to-use device, it can be coupled to any type of equipment, observing the guidelines for its proper use.

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The Triple Syringe is a peripheral equipment used in dental procedures, intra and / or extra buccal, indicated in cleaning, drying, irrigation and moistening, either through air, water or the combination of both (Spray).

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The Triple Syringe can be disinfected with alcohol 70° GL, applied on its surface through a cloth or spray.

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It is recommended to disinfect the Triple Syringe routinely between each service.

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Autoclavar apenas a ponta da Seringa Tríplice a 135°C.

Only autoclave the tip of the Triple Syringe at 135 ° C.

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Do not clean the equipment using ultra sonic cleaners

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The equipment must be sterilized before and after each treatment. Pack the product in packaging for sterilization and dispose only in an autoclave at a maximum temperature 135° C and 2.2 Bar.

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Sterilize the equipment before using it for the first time. Use compressed dry air, filtered and with no oil.

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Check that the hoses are securely fastened so there is no leakage.

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*Keep in a place protected from the sun, heat, dust and humidity;
Keep away from chemicals and cleaning agents;
Carry out the transport only in its respective packaging;
Ambient temperature (transport and storage): -10°C to 55°C;
Relative air humidity (transport and storage): 0% to 85%;
Atmospheric pressure: 70kPa to 106kPa;
Maximum stacking: 8 units.*

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1 - The Triple Syringe buttons have different functions. When pressuring the lower button releases the air passage of the pneumatic system. In the upper button's case, its activation provides the water release, as shown in Fig. 1.

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2 - Pressing the two buttons simultaneously will cause Spray formation as shown in Fig. 2.

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1 - Connect the tip to the body of the Triple Syringe as indicated (A).
To remove the tip do the movement in the opposite direction (B). (Fig. 3)

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2 – Unscrew the cable from the Triple Syringe (Fig. 4), pass the double hose used to transport air and water through the inner part of the cable and connect it to their respective connectors (Fig. 5).

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Fig. 5 - Hoses' connection to the entrance connectors

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3- Screw the Triple Syringe cable, thus finalizing the operation.

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Like any other electromedical device, Triple Syringe is composed of several materials, such as metals, rubbers, etc. Therefore, in order to avoid environmental contamination or the improper use of the equipment after its destruction, it must be disposed of in an appropriate place (following local legislation for waste disposal conditions).

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Dent-Flex Ind. E Com. Ltda: It declares that the materials used in its Applied Parts (according to the NBR IEC 60601-1 standard) in Triple Syringe have been widely used in the dental field over the years, guaranteeing its biocompatibility. In addition, the consumable items of the equipment are: distilled water and air.

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All recommendations for use are found in this Manual, but if any problems or malfunctions are detected please contact the Dentflex Technical Support Center to correct the problem. Do not attempt to open the product and / or repair it yourself or with help of persons not trained / authorized to do so, this may aggravate the problem or even generate faults that could compromise product safety.

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SENSITIVITY TO PREDICTABLE ENVIRONMENTAL CONDITIONS NORMAL USE SITUATIONS

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The Triple Syringe was designed to have a maximum noise level of 80 dB, as long as it is installed, kept clean and maintained, transported and operated according to these use instructions.

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For warranty's information see the certificate of the product. The warranty's product covers manufacture defects counted from the date of sale to the first client or the delivery of the warranty certificate. The warranty becomes invalid in case any of the following situations is verified:

- Installation and/or operation in disagreement with this manual;
- Fall;
- Damages to the external finishing and/ or accessories whose the manufacturer has not given any cause.

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E-mail: sac@dentflex.com.br / Telefones (16) 99404-4435 e (16) 2133-0000.

With the purpose of offering a service that guarantees 100% satisfaction, we present to the customer the Dentflex Technical Support Center, an innovation in the service that came to provide agility, safety and transparency in the maintenance process of your Dentflex product. We have an infrastructure and a qualified professionals' team, guaranteeing the most complete technical assistance for the entire Dentflex product line.

- Trained technicians and engineers;
- Last generation machinery and equipment;
- Original components.

To use the service offered by the Dentflex Technical Support Center, please, contact us: E-mail: sac@dentflex.com.br / Phones: + 55 (16) 9 9404-4435 and + 55 (16) 2133-0000.

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